

Installation Instructions

Tel: 925.427.7800
Fax: 925.427.0800

support@HSIheadwalls.com



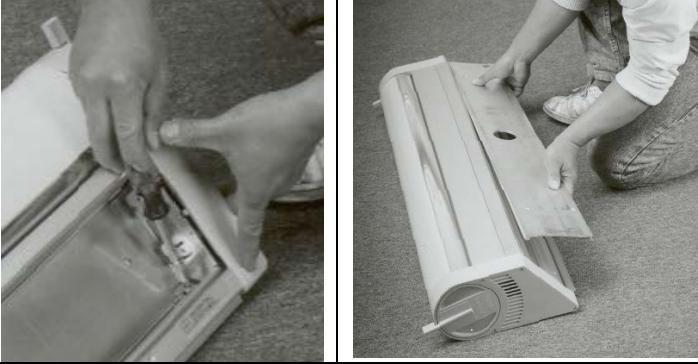

AURORA™ LIGHT FIXTURE



READ THROUGH THESE INSTRUCTIONS COMPLETELY PRIOR TO
ATTEMPTING ANY INSTALLATION

Room finishing should be completed and a junction box with appropriate wiring to the correct circuit should be installed at the proper fixture location before installing the Aurora™ light fixture

Hospital Systems Inc.
750 Garcia Avenue
Pittsburg CA 94565
USA

<p>1. Remove the lower lens by unscrewing the two lens screws</p>	
<p>Remove the mounting plate shipped attached to the Aurora™ Light at the back of the fixture with two safety nuts located inside the lower lamp cavity.</p> <p> DO NOT REMOVE THE SPRINGS.</p>	
<p>Wire the provided connector to the power feed. Make sure the circuit is OFF prior to wiring this connector. Install the mounting bracket using appropriate hardware for the type of backer utilized inside the building wall. Or attach to appropriate backer in headwall module</p>	
<p>Make sure to feed- the connector through the hole at the center of the mounting plate. NOTE: This mounting plate must be level and completely tight against the building wall, and must not be bent or bowed.</p>	

Position the fixture so that it rests in the top groove of the mounting plate. Tilt the fixture up and connect the mating plug with the connector at the back of the fixture. Lower the fixture and align it-on the safety screws. **BE SURE THE SPRINGS ARE IN PLACE.**



Re-install the safety nuts. (Do not over tighten. You may need to adjust the safety nuts supplied. Use of any other nut may allow the fixture to fall.



Install the appropriate lamps (not included) and replace the lenses and screws. Test the optional safety feature, if included, and make final adjustments as necessary.



If you have any questions regarding any of the above, please contact your local representative, or the factory direct at 925.427.7800

